

Customer Bulletin – January 2017

Revised Range Card: PTS Panels® Multi-Chemistry Controls (Lot Number MC22) – (Cat. No. / REF 721)

Summary:

A new range card has been issued for PTS Panels Multi-Chemistry Controls (LOT Number MC22). No other lots are affected.

Single Analyte Glucose Range Update (including PTS Panels® eGLU test strips):

The new range card updates the range for glucose when utilizing single-analyte PTS Panels test strips. The updated ranges are provided below, with the change highlighted. Updated range cards are also available at www.ptsdiagnostics.com. The previous, obsoleted range cards are also provided for reference.

PTS Panels Multi-Chemistry Controls – Lot Number MC22

PTS Panels® Multi-Chemistry Control* Ranges	Level 1		Level 2	
	mg/dL	mmoL/L	mg/dL	mmoL/L
Lot #: MC22 Expiration Date: 2017 03 31				
Single-Analyte Test Strips				
Cholesterol	120 - 220	3.10 - 5.69	180 - 340	4.65 - 8.81
Triglycerides	90 - 170	1.02 - 1.92	160 - 260	1.81 - 2.94
Glucose/eGLU™	50 - 120	2.78 - 6.66	200 - 360	11.10 - 19.98
Ketone	≤2 - 20	≤0.19 - 1.92	20 - 60	1.92 - 5.77
Multi-Analyte Test Strips				
Cholesterol	110 - 220	2.84 - 5.69	160 - 280	4.14 - 7.24
Triglycerides	70 - 190	0.79 - 2.15	140 - 270	1.58 - 3.05
Glucose	30 - 90	1.67 - 5.00	140 - 340	7.78 - 18.89
*HDL Cholesterol requires a separate HDL Cholesterol Control (Cat. No. 722). For assistance contact PTS Customer Service @ 877-870-5610 or Fax @ 317-870-5608. Outside of US @ 317-870-5610. Email: inforequest@cardiochek.com				
				PS-002519 Rev. 9

Previous Range Card Information (Obsolete):

PTS Panels Multi-Chemistry Controls - Lot Number MC22 - OBSOLETE

PTS Panels® Multi-Chemistry Control* Ranges	Level 1		Level 2	
	mg/dL	mmoL/L	mg/dL	mmoL/L
Lot #: MC22 Expiration Date: 2017 03 31				
Single-Analyte Test Strips				
Cholesterol	120 - 220	3.10 - 5.69	180 - 340	4.65 - 8.81
Triglycerides	90 - 170	1.02 - 1.92	160 - 260	1.81 - 2.94
Glucose/eGLU™	50 - 110	2.78 - 6.11	200 - 310	11.11 - 17.22
Ketone	≤2 - 20	≤0.19 - 1.92	20 - 60	1.92 - 5.77
Multi-Analyte Test Strips				
Cholesterol	110 - 220	2.84 - 5.69	160 - 280	4.14 - 7.24
Triglycerides	70 - 190	0.79 - 2.15	140 - 270	1.58 - 3.05
Glucose	30 - 90	1.67 - 5.00	140 - 340	7.78 - 18.89
*HDL Cholesterol requires a separate HDL Cholesterol Control (Cat. No. 722). For assistance contact PTS Customer Service @ 877-870-5610 or Fax @ 317-870-5608. Outside of US @ 317-870-5610. Email: inforequest@cardiochek.com				
PS-002519 Rev. 9				

Questions/Comments?

Please call Customer Service at +1-317-870-5610 (direct) or 1-877-870-5610 (toll-free US only).

CB 16-009 r0 01/17