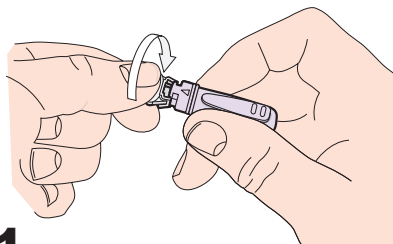
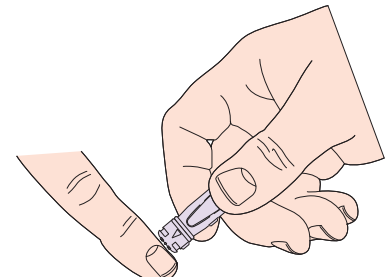
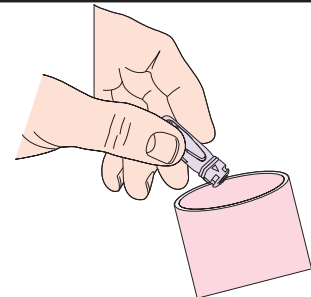


How to Use Unistik® 3 Pre-set Single Use Safety Lancets

| | | |
|--|--|--|
|  <p>1</p> |  <p>2</p> |  <p>3</p> |
| <p>Twist off and discard lancet cap. DO NOT PULL.</p> <p>Tuerza hacia afuera y descarte la tapa de la lanceta. NO TIRAR.</p> <p>Kappe abdrehen und entsorgen. NICHT ZIEHEN.</p> <p>Dévissez et jetez le capuchon. NE PAS TIRER.</p> <p>Ruotare il cappuccio e provvedere allo smaltimento dello stesso. NON TIRARE.</p> <p>Desenrosque e descarte a tampa da lanceta. NÃO PUXAR.</p> <p>Draai het lancetkapje eraf en gooi het weg. NIET AAN TREKKEN.</p> | <p>Press platform firmly against chosen site and press release button.</p> <p>Coloque el extremo del Unistik 3 apoyado en el dedo y prosiga a presionar la almohadilla activadora superior.</p> <p>Unistik 3 gegen Finger drücken-auslösen.</p> <p>Posez Unistik 3 sur le bord extérieur du doigt. Appuyez sur le déclencheur... clic ... Unistik 3 est déclenché. La lancette se rétracte automatiquement à l'intérieur.</p> <p>Premere con fermezza la piattaforma contro il punto di prelievo prescelto e premere poi il pulsante di rilascio.</p> <p>Pressione a plataforma firmemente contra o local seleccionada e accione o botão de disparo.</p> <p>Druk de platte kant stevig tegen de gekozen plaats en druk op de ontspanknop.</p> | <p>Dispose of in a suitable sharps container.</p> <p>Deseche el Unistik 3 en un receptáculo de agujas adecuado.</p> <p>Unistik 3 nach Gebrauch sachgerecht entsorgen.</p> <p>Jetez Unistik 3 dans un conteneur à aiguilles usagées.</p> <p>Disporre negli appositi contenitori.</p> <p>Descarte num recipiente apropiado para o efeito.</p> <p>Wegwerpen in een daarvoor geschikte scherpaafvalcontainer.</p> |